

Artis Dialysis Fluid preparation

Dialysis fluid preparation

Temperature 35 to 39.5 °C
 Flow rate 300 to 800 ml/min, in steps of 50l/min (± 2% of the set value)
 [Na+] range 130 to 160 mmol/L (± 2.5%)
 [HC03-Jrange 24 to 38 mmol/L (± 5%)

K-1.5 Central Delivery System		K-1 Jug		K-2 Jug	
To achieve Na level of:	HC03 level must be adjusted to:	To achieve Na level of:	HC03 level must be adjusted to:	To achieve Na level of:	HC03 level must be adjusted to:
134 to 140	24 to 38	136 to 140	24 to 38	134 to 140	24 to 38
133	less than 36	135	less than 36	133	less than 36
132	less than 33	134	less than 33	132	less than 33
131	less than 30	133	less than 30	131	less than 30
130	less than 27	132	less than 27	130	less than 27
		131 & 130	not possible		
K-3 Central Delivery System		K-1.5 Jug		K-3 Jug	
To achieve Na level of:	HC03 level must be adjusted to:	To achieve Na level of:	HC03 level must be adjusted to:	To achieve Na level of:	HC03 level must be adjusted to:
135 to 140	24 to 38	135 to 140	24 to 38	133 to 140	25 to 38
134	less than 37	134	less than 37	132	less than 36
133	less than 34	133	less than 34	131	less than 33
132	less than 31	132	less than 31	130	less than 31
131	less than 28	131	less than 28		
130	less than 25	130	less than 25		